
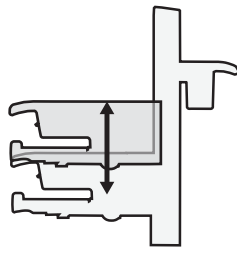


Warning







⚠

1. Moving Parts without Advanced Notice Can Result Crush and Personal Injury!
Be cautious before:
 - Removing Mounted Facilities on the Cart
 - Packaging the Cart
 - Restoring the Cart
2. Lower the working desk to the lowest position and fixed it.
3. Failure to follow this instruction shall result serious personal injury or property damage.

N2 Medical Cart - Dual Panels Supported - Pole Mount

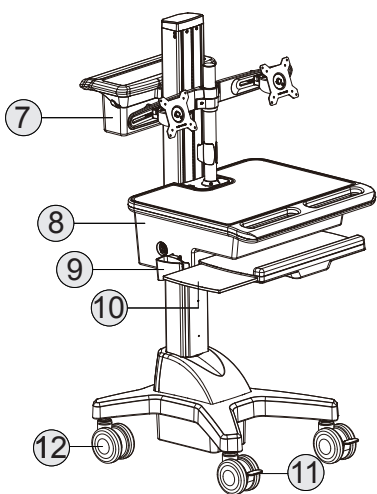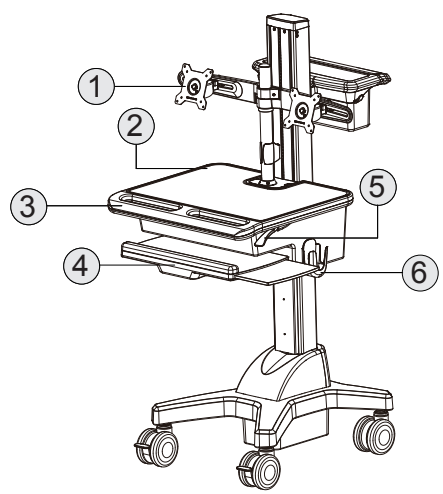

1. Dual-Panel Support Module with Hinge Adaptor
Compliant with VESA 75/100 mounts patterns
2. Working Surface with embedded compartment
3. Front Handle
4. Wrist Comfort
5. Lift Lock/Release
6. Mouse Holder
7. Storage Bin & Rear Handle
8. Storage Compartment
9. Scanner Holder
10. Keyboard Tray (can be tilted)
11. Front Caster with Brake
12. Rear Caster

load 1. On the working surface: max. 7 kgs
2. Inside the compartment: max. 5 kgs

Height Adjust

205mm

500mm

load Max. 1.5 kg

Keyboard Tray

200mm

12° (Tilt-Back)

Display System

size ≤24"

Rotate 360°

load Max. 10 Kg (Total)

Torque Adjust + -20°

Mouse Pad +/-20°



Temperature Range

Operational	Storage
Temperature Range: 10°C / 50°F - 29°C / 86°F Relative Humidity Range: 5-95% rH	Temperature Range: 10°C / 50°F - 20°C / 68°F Relative Humidity Range: 5-95% rH





Warning:
An external CPU Holder shall be required for CPU install when the total weight of CPU and Panels together > 28.6 lbs (13 kgs).

Torque Adjust


Tilt +/-20°

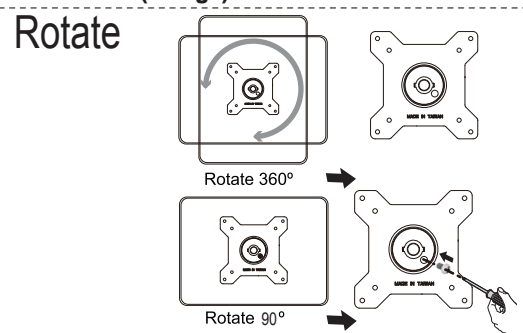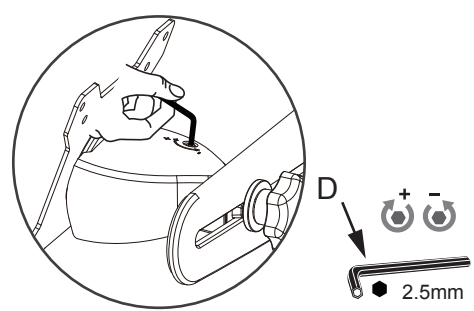


Dimensions

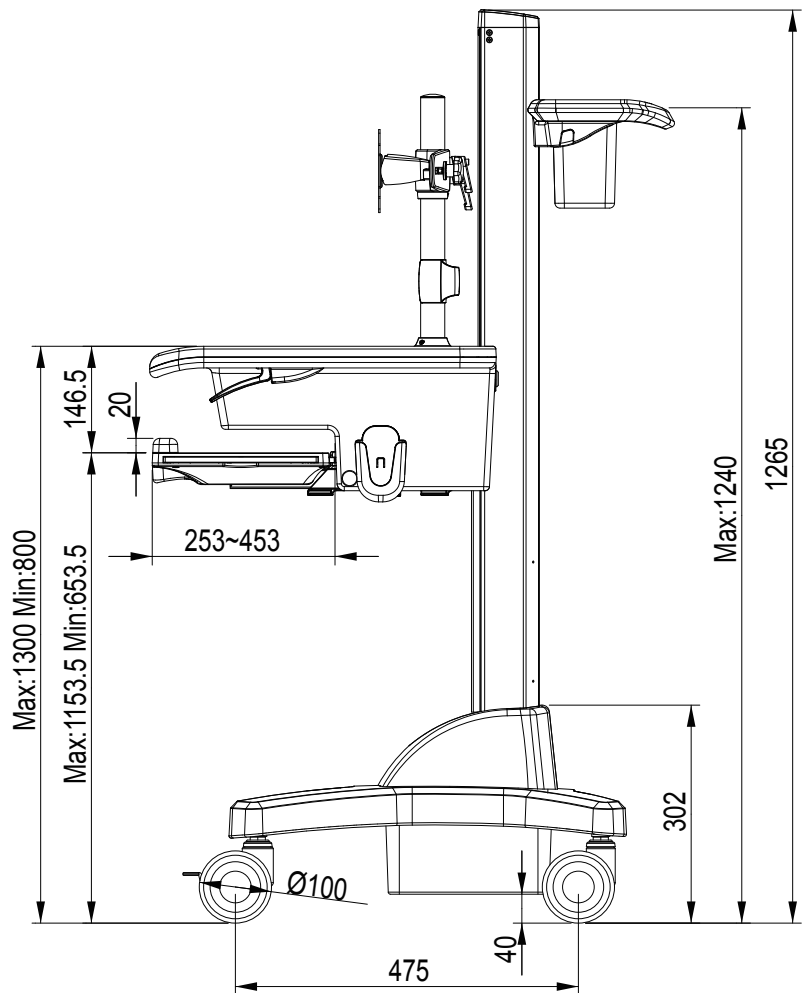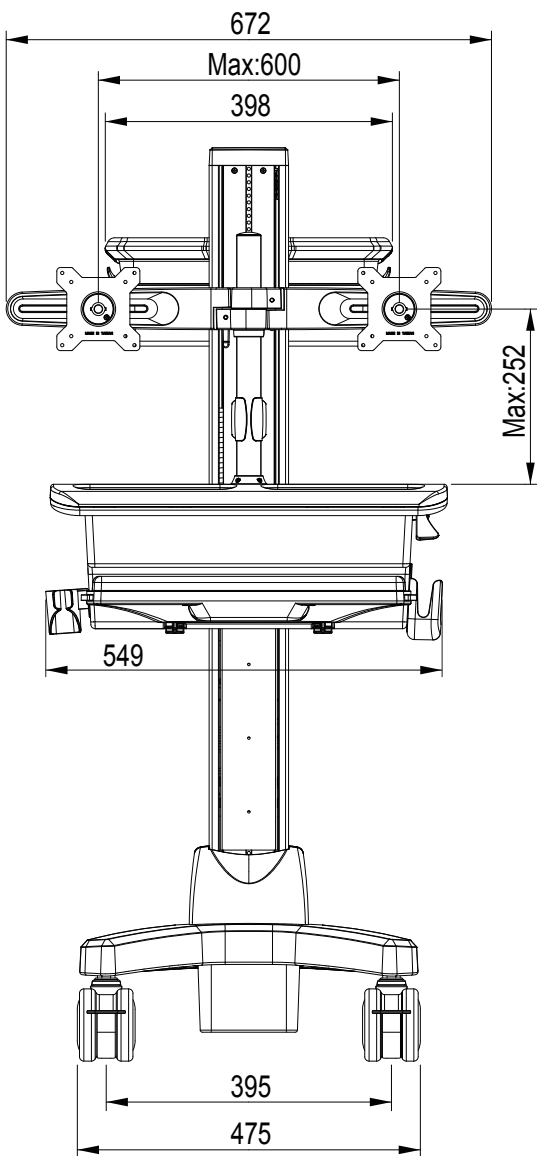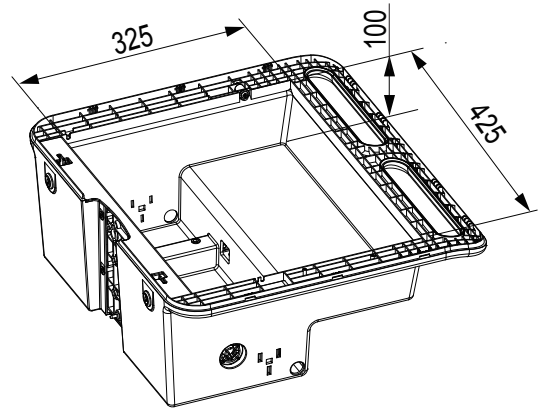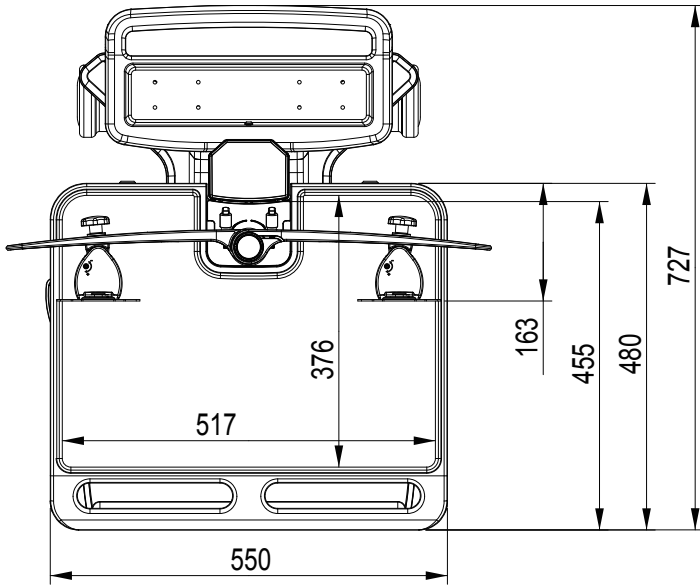


Packging Parts







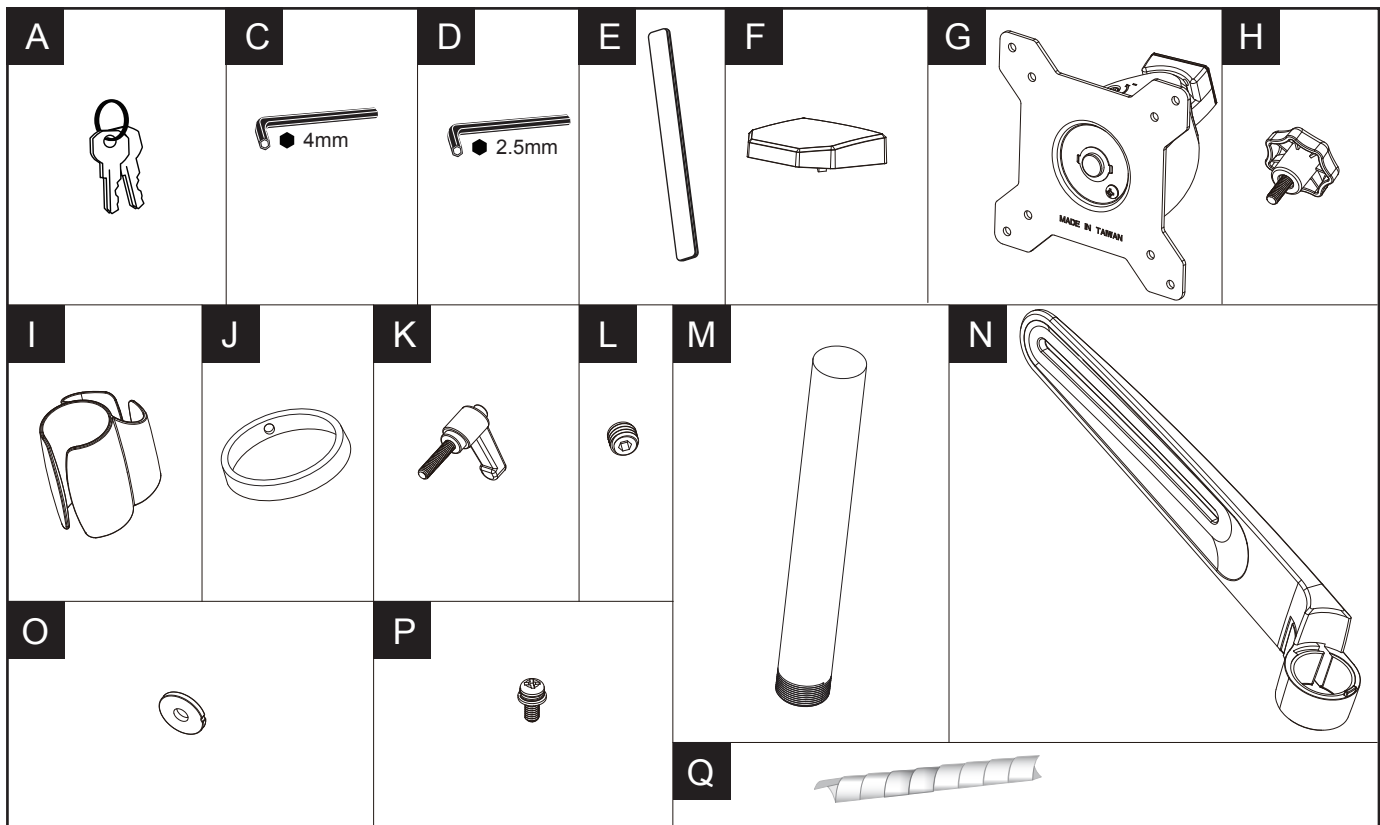









Packging Parts

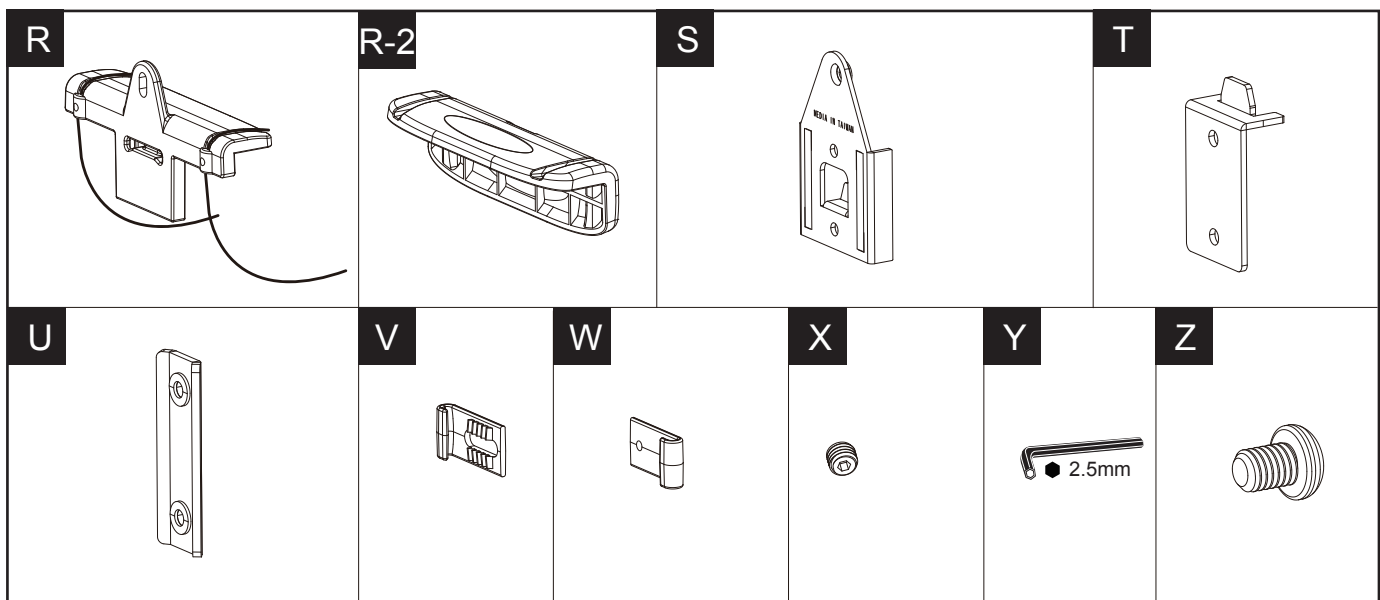


Required Tools

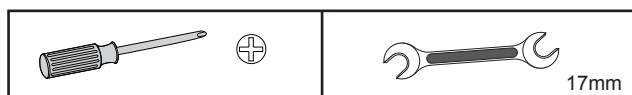

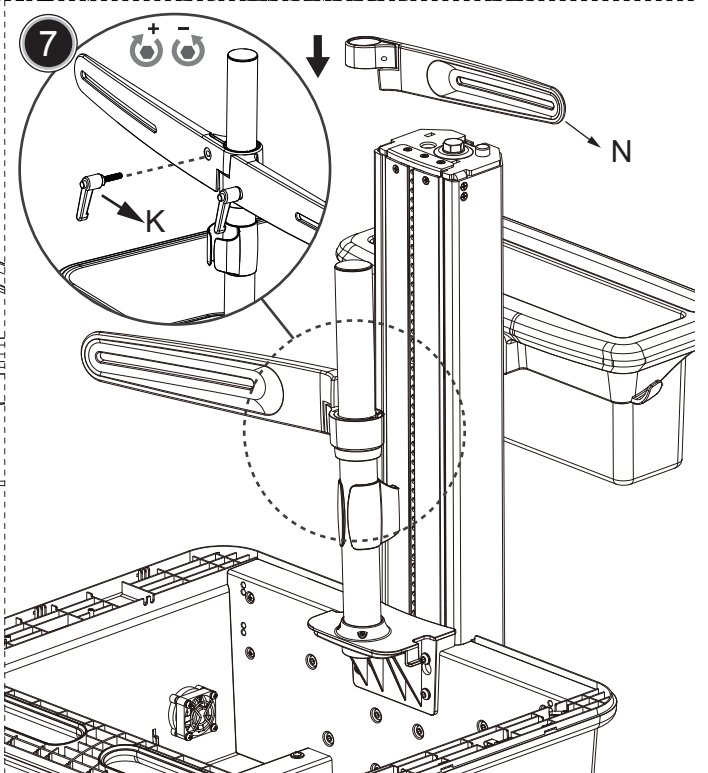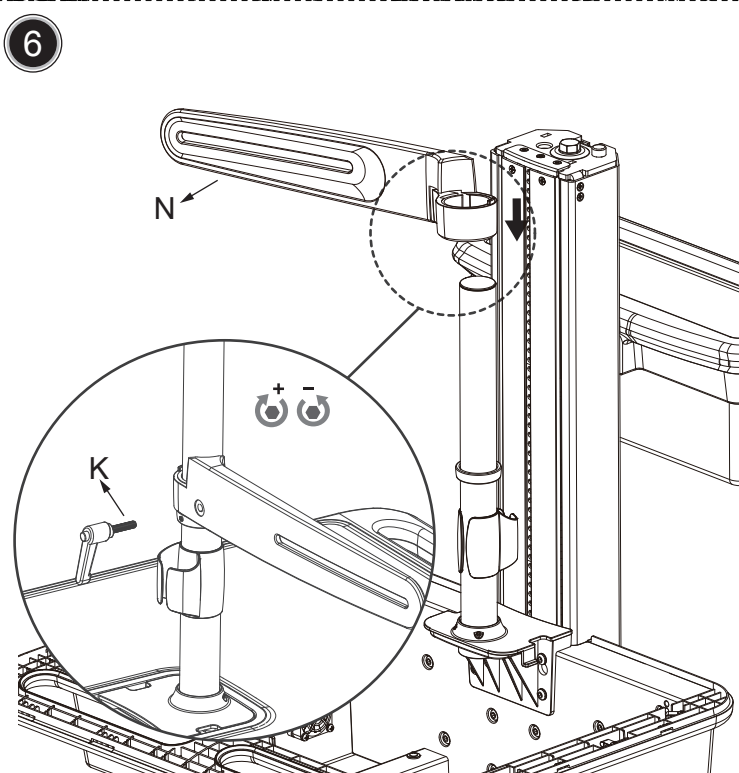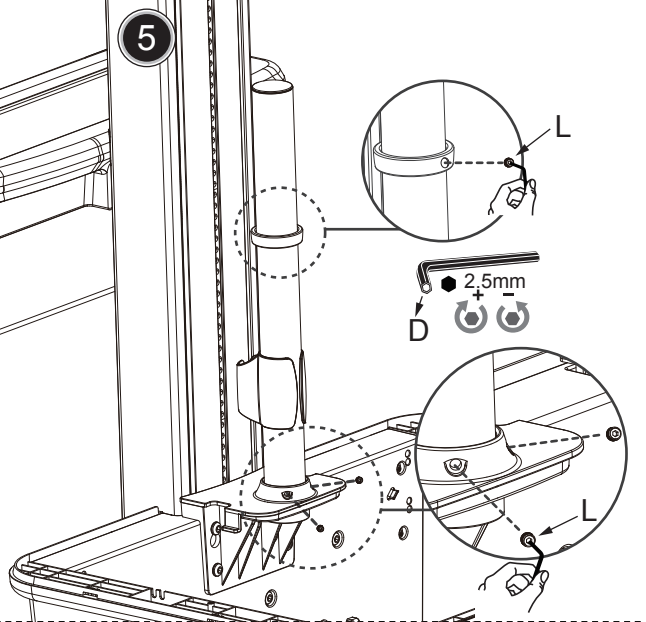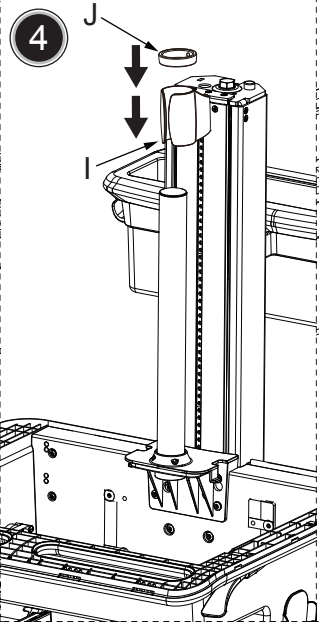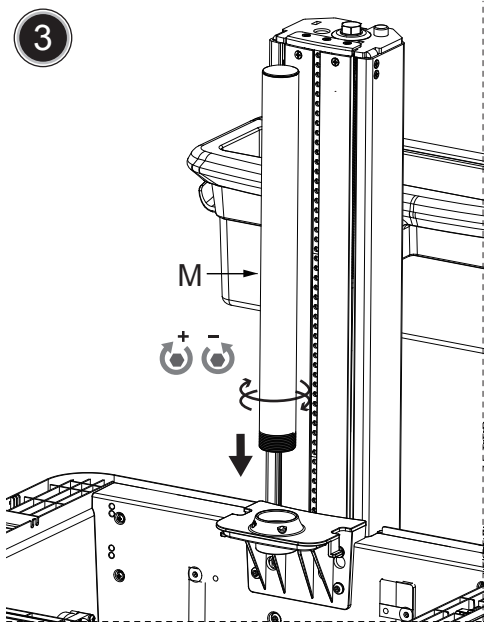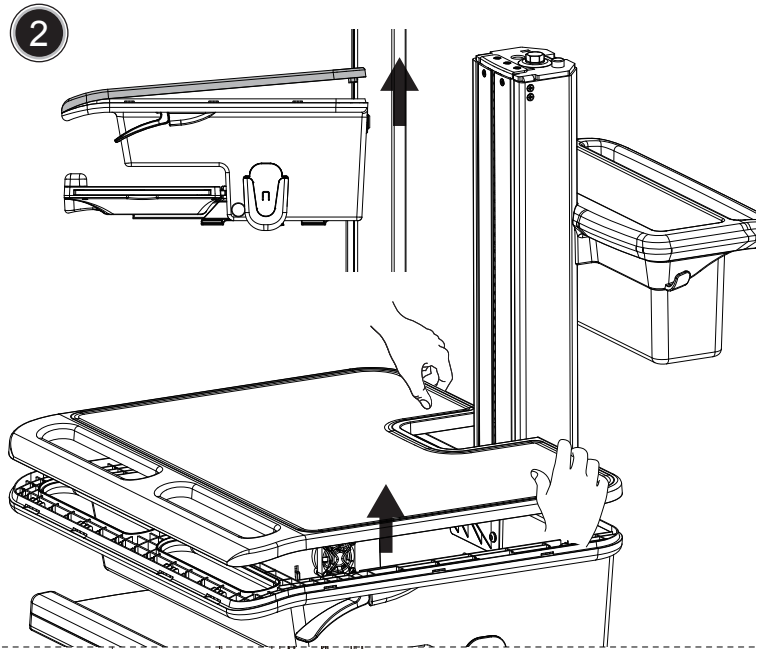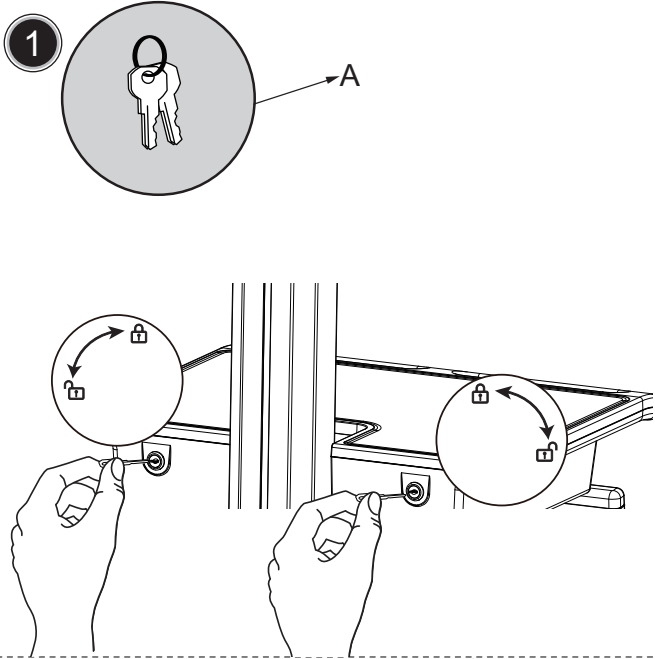


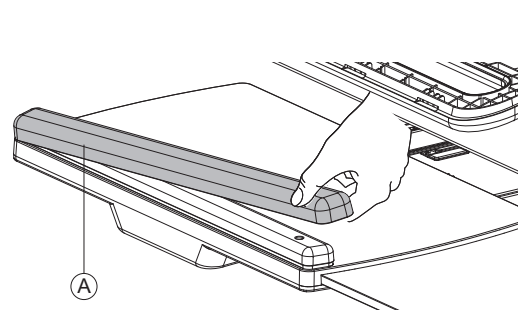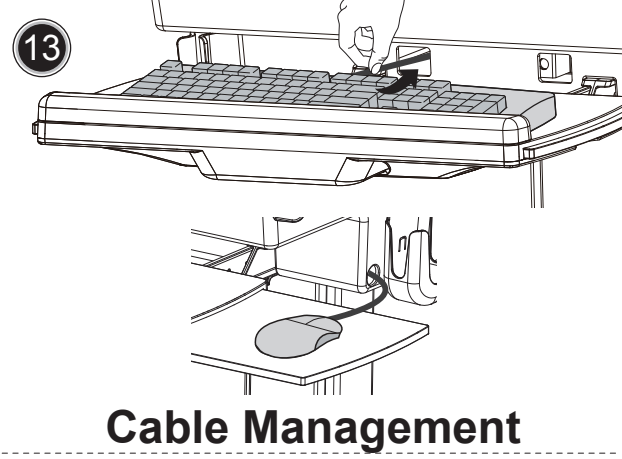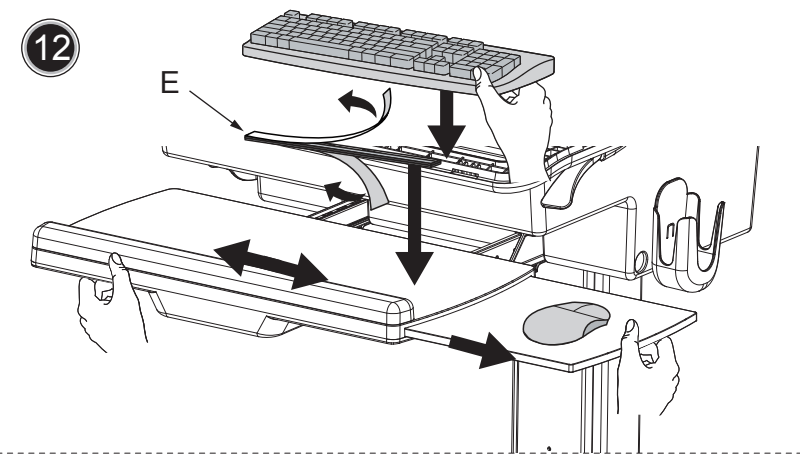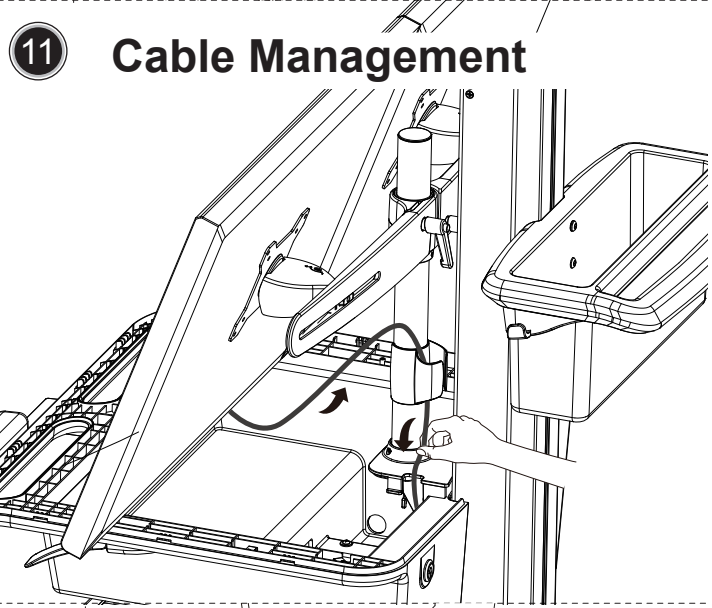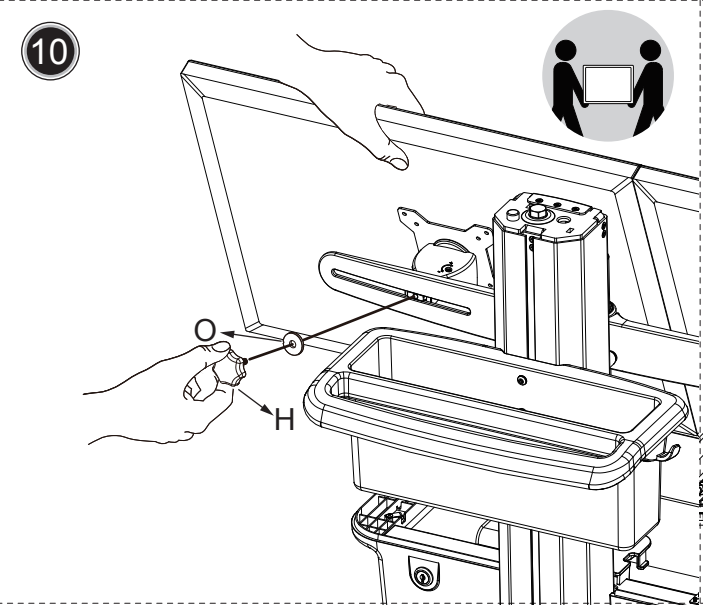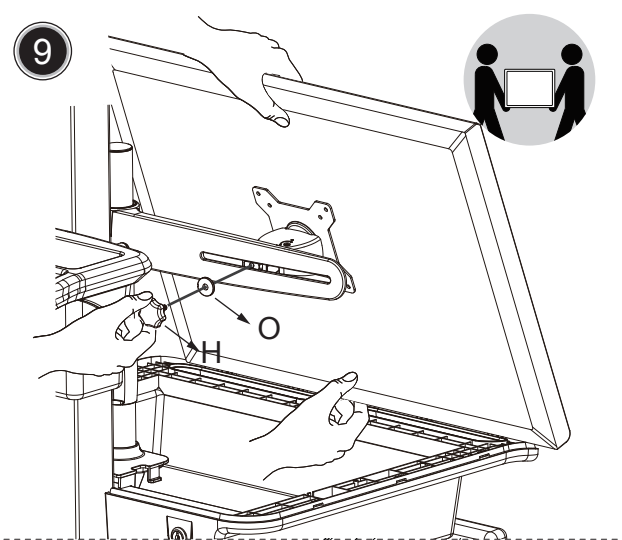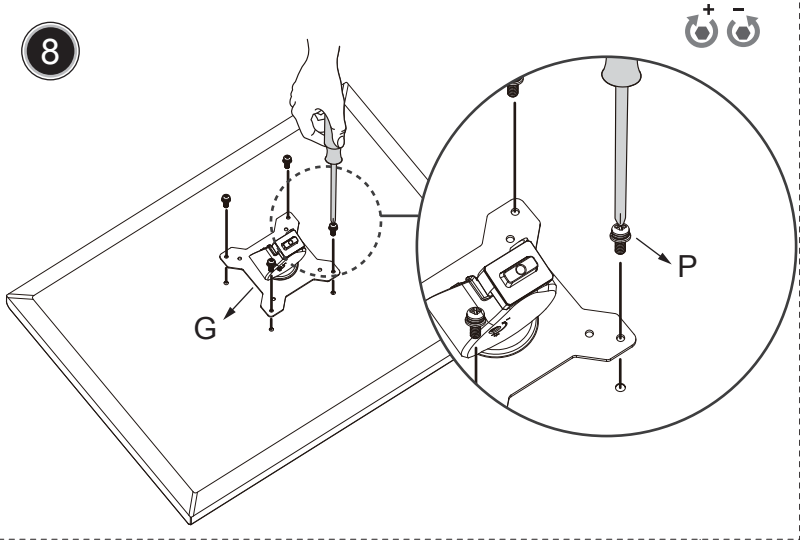



Rubber (A) can be taken off for essential cleaning

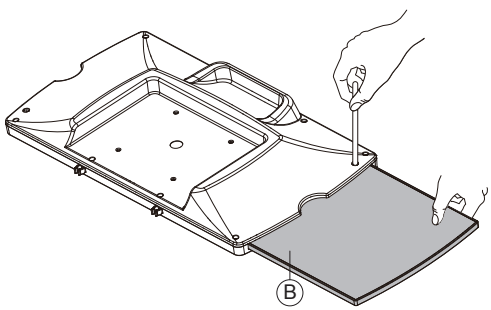

Pad board (b) can be pulled out for cleaning








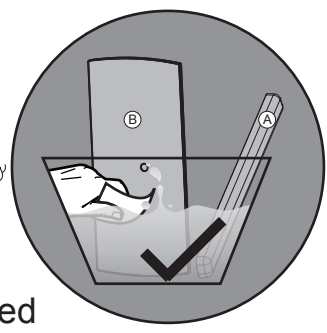








13

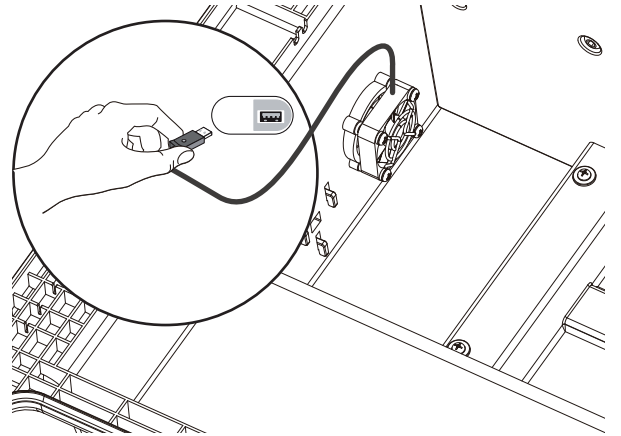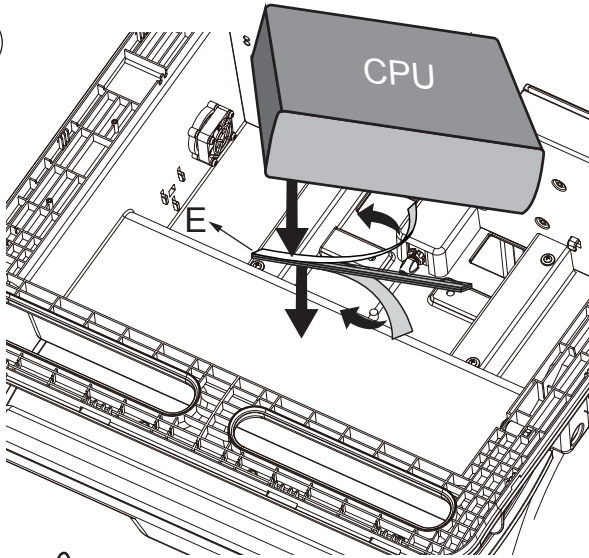







Don't Block the Fan.
Block the Fan shall result high heat inside the compartment and damage facilities located there.

Accessory PC Holder

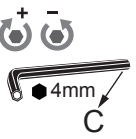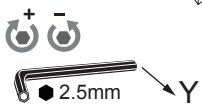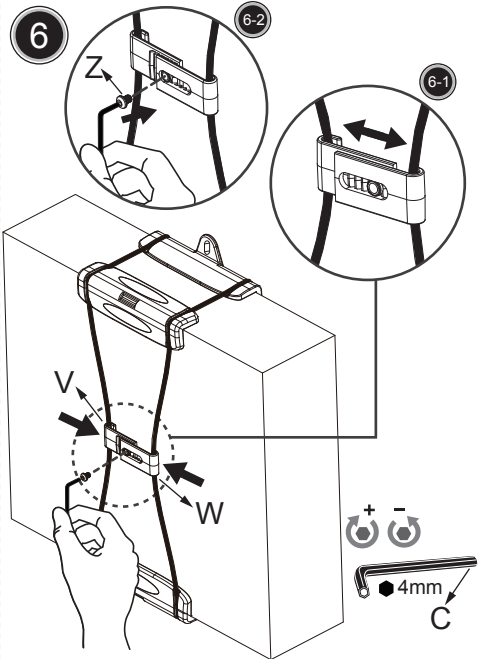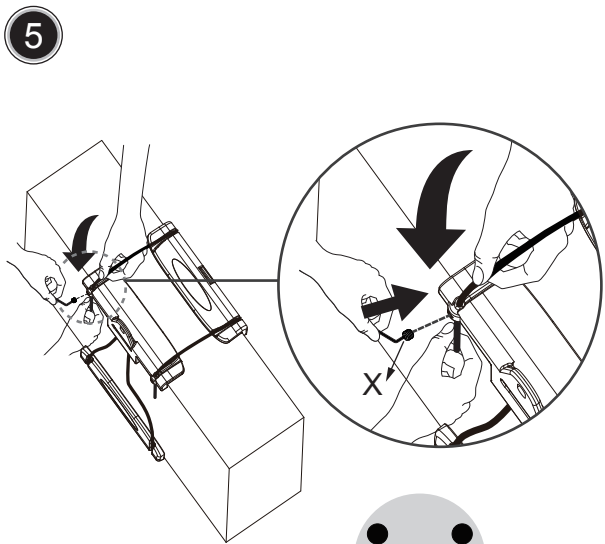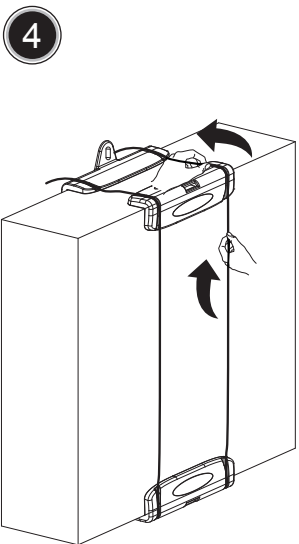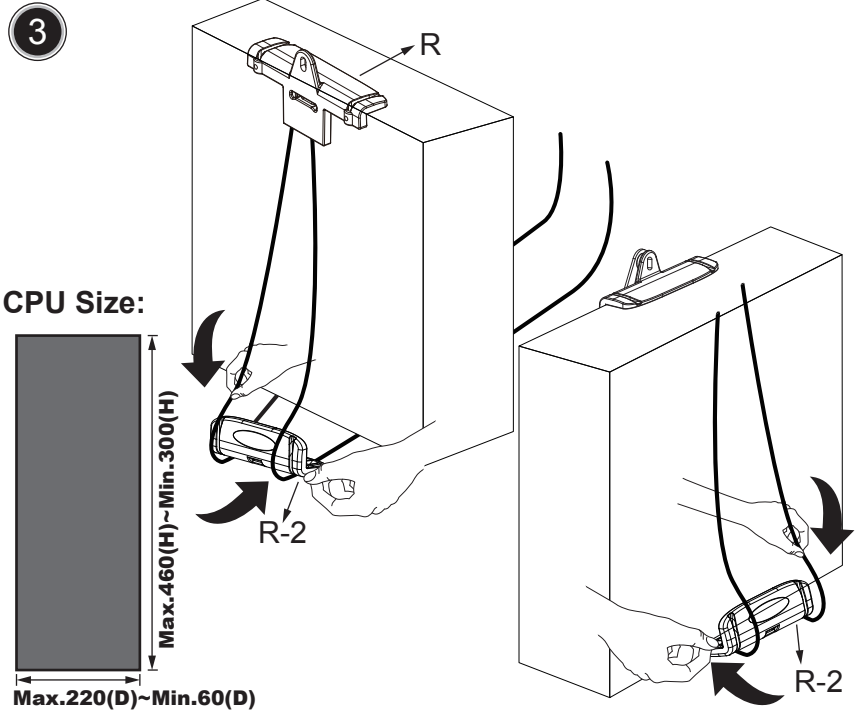
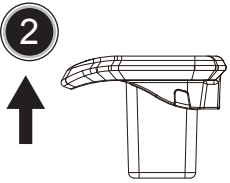
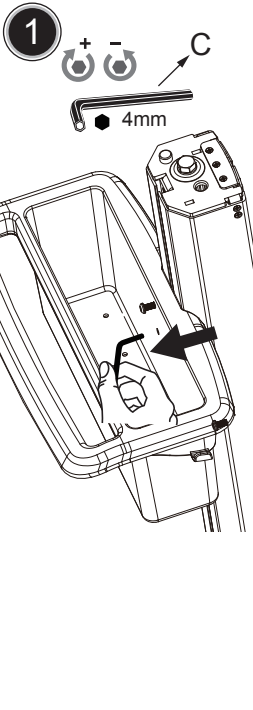


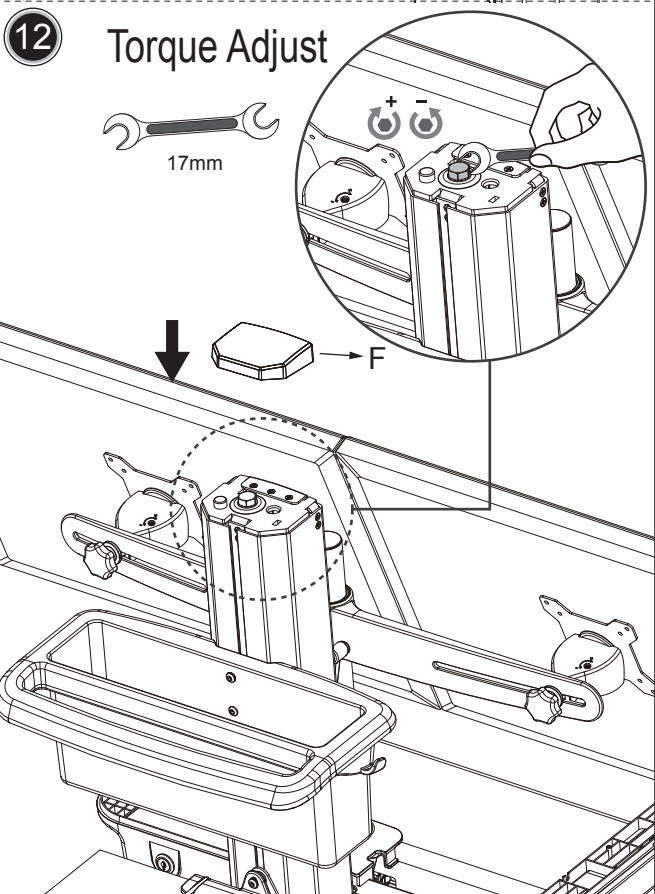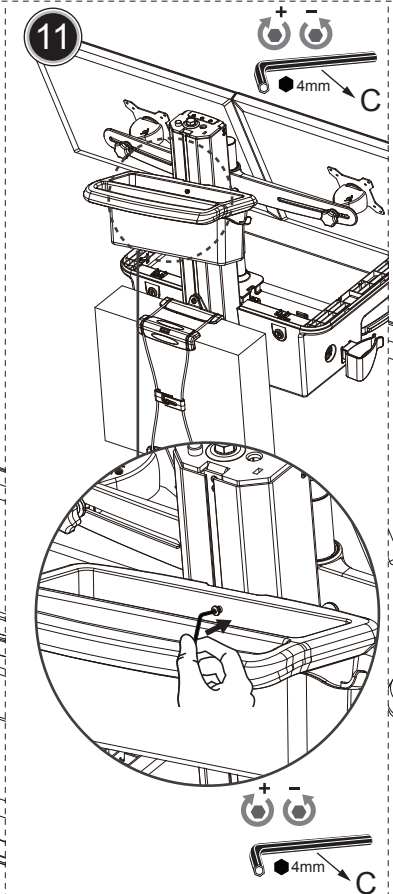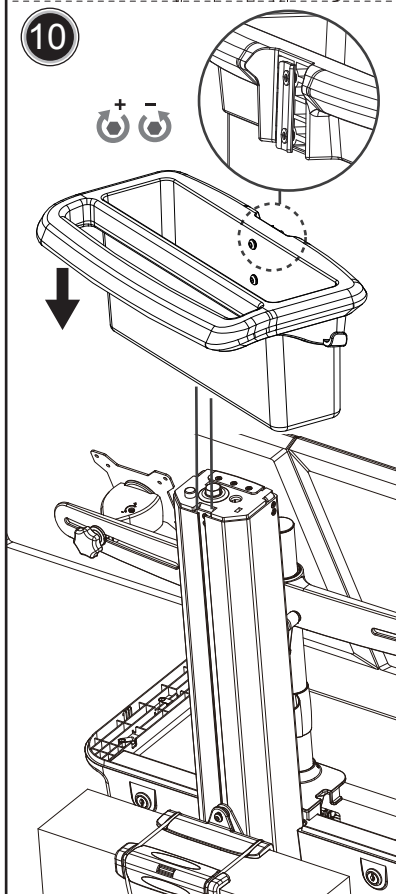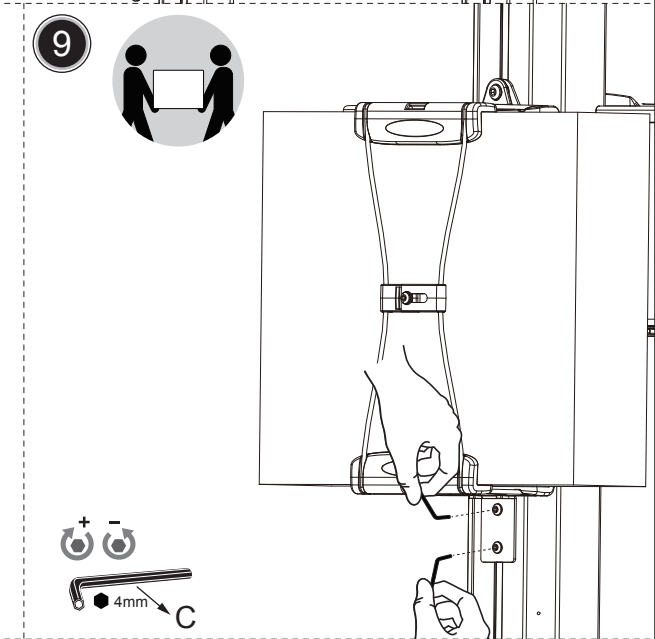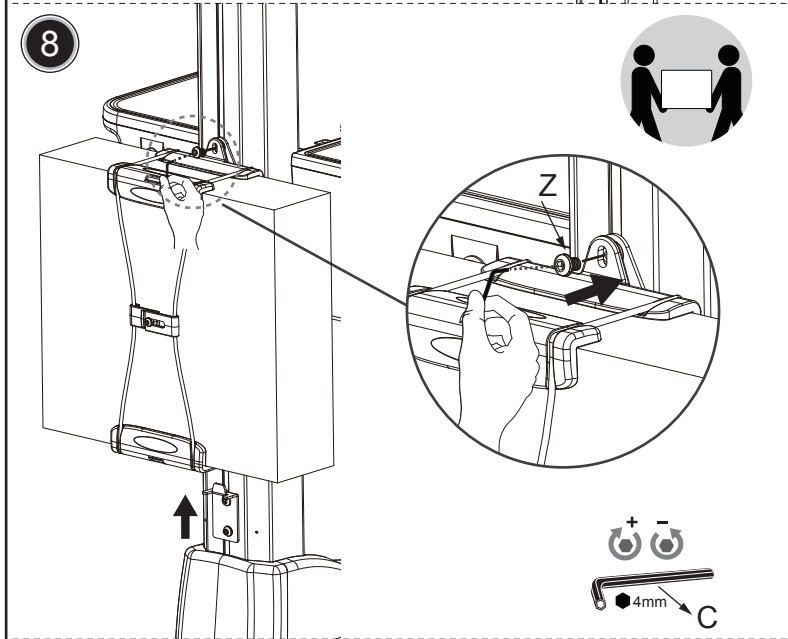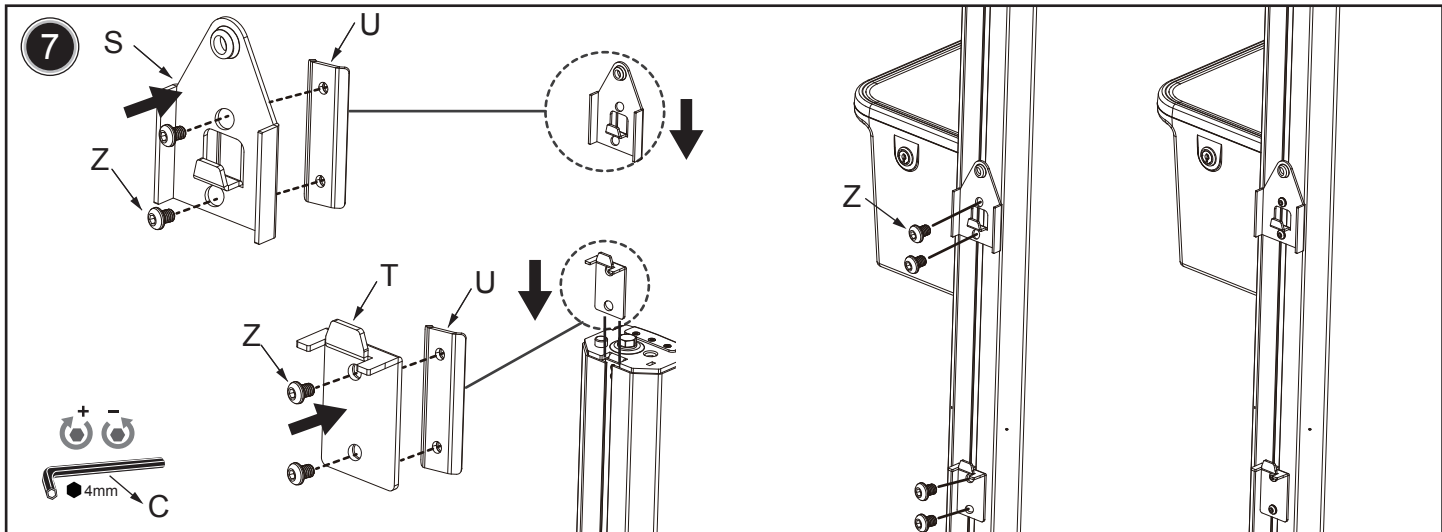

Cable Management

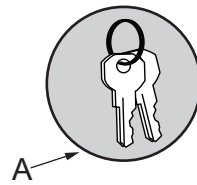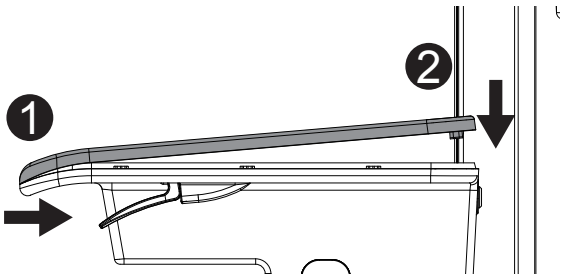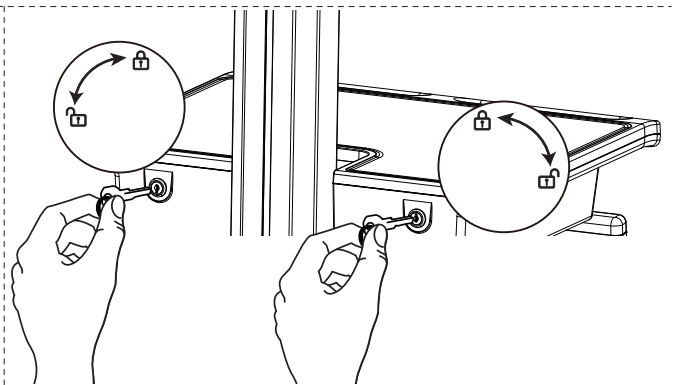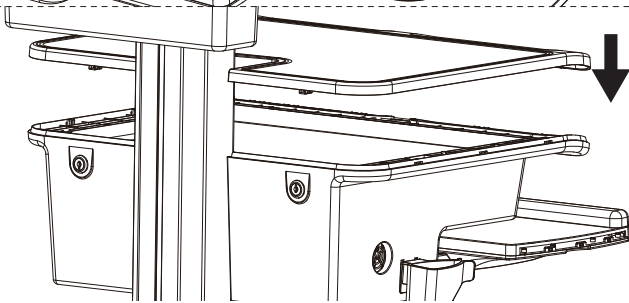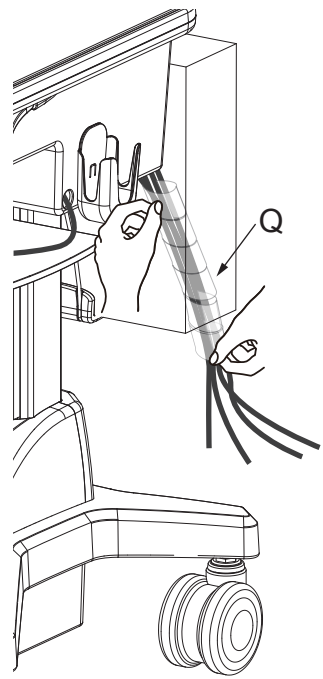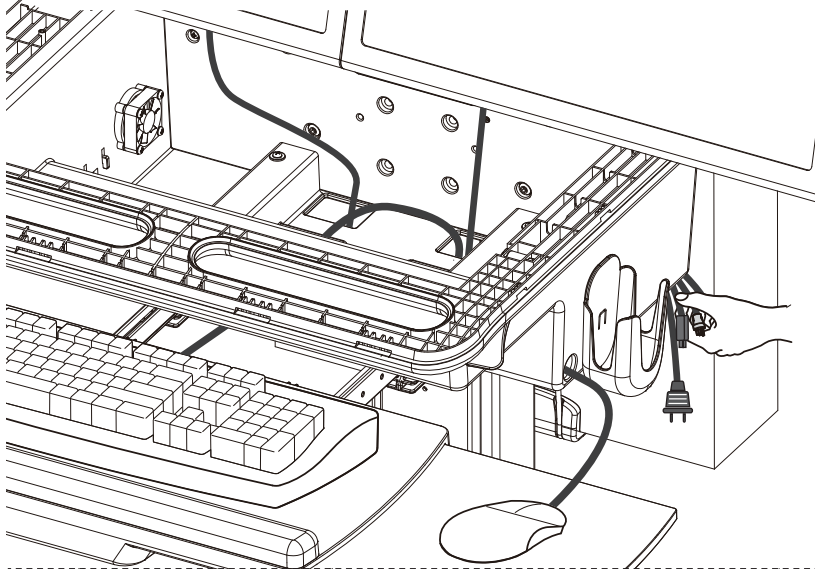

⚠ Lock Before Adjust The Height Of Lift

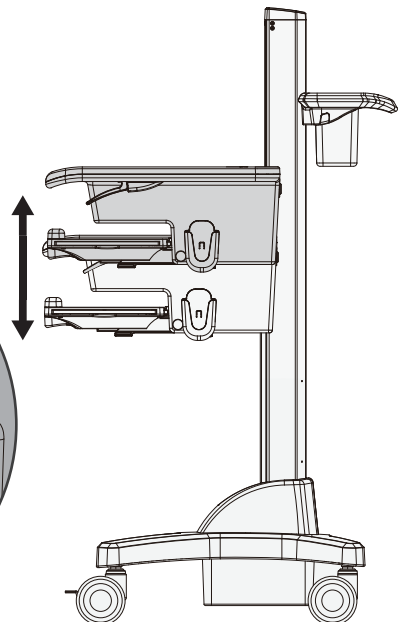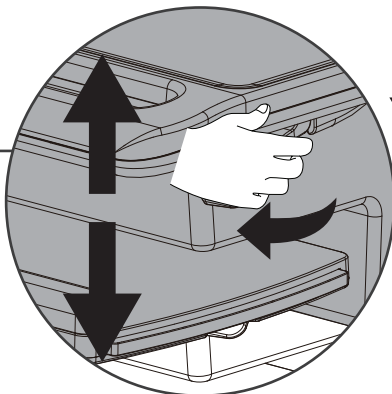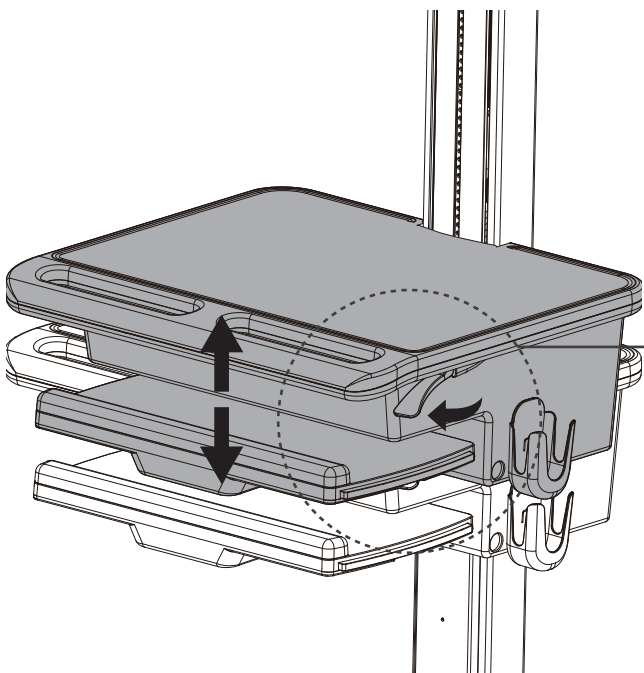


Ergonomics

Working

Customized to your height

- A. Set the top of monitor one inch lower than your eye level.
- B. Tilt the monitor for comfortable viewing angle to reduce eye strain and fatigue at neck.
- C. Pull the keyboard tray forward & position the mouse pad (right or left) at your needs.
- D. Work with elbows bent at 90° so to minimize the muscle strain.
- E. If the compartment desk moves up and down difficultly, or it drifts from set position, please consult the adjustment information listed at this manual.

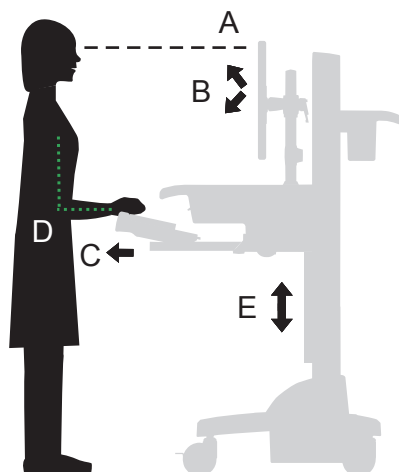

Ergonomics

Moving

Stow the cart – before you go.

- A. During normal movement, release the lift brake and lower the compartment desk to the lowest position. This process shall increase the stability on move and maintain an unobstructed view.
- B. Tuck tray back and have all those accessories fixed at right positions.
- C. Unlock the front casters.
- D. Push cart forward from rear handle with elbows bent at 90° so to maximize the control and minimize muscle strain.
- E. Don't run out on the way!!

